DONJOY[®]

ISOFORM™ POSTURAL
EXTENSION BRACE

ENGLISH 2
ESPAÑOL..... 4
DEUTSCH 6
ITALIANO 8
FRANÇAIS..... 10
NEDERLANDS 12
PORTUGUÊS..... 14
ČEŠTINA 16
SLOVENČINA 18
DANSK..... 20
SVENSKA..... 22
SUOMI 24
NORSK..... 26
简体中文 28
日本語 30

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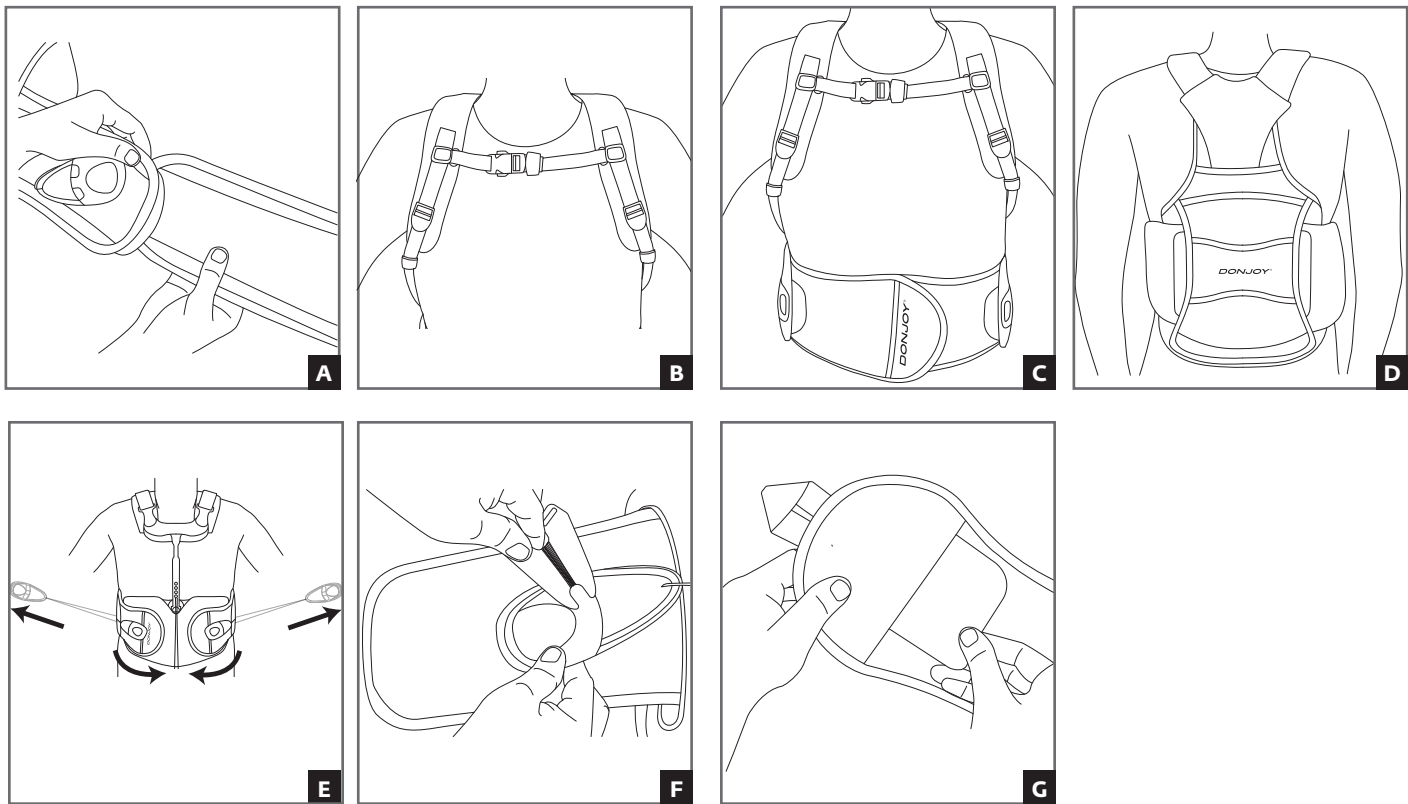
DJO, LLC
1430 Decision Street
Vista, CA 92081-8553 • USA

ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE: The intended user should be a suitably trained professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

INTENDED USE/INDICATIONS: The IsoFORM Postural Extension brace is designed to support and stabilize the spine and may be suitable for people with thoracic spine conditions such as kyphosis, osteoporosis, acute pain, mechanical back pain, compression fractures, strain and mild scoliosis.

PERFORMANCE CHARACTERISTICS: Soft-good/semi-rigid designed to restrict motion through elastic or semi-rigid construction. Providing immobilization or controlled movement of the limb or body segment.

CONTRAINDICATIONS: Pregnancy and circulation, pulmonary, cardiovascular or skeletal conditions which pose risk to be made worse as a result of compression of the abdomen and/or pressure, unstable, displaced fractures. Do not use if you are allergic to any of the materials contained within this product.

WARNINGS AND CAUTIONS: If you experience any pain, swelling, sensation changes, excessive discoloration, irritation to the skin or any unusual reaction while using this product, consult your medical professional. If any distortion of the support panels, fraying or deterioration of lace cables is noticed, discontinue use. Brace is intended to be worn over clothing. Brace should be appropriately sized to the patient to ensure proper support and function. If the brace does not fit properly, it may be adjusted by a healthcare professional. Care should be taken with patients, especially the elderly, not to apply the brace too tightly, which MAY cause excessive pressure. Patients should be instructed on appropriate tensioning during ongoing use. Patient should see a healthcare professional for a replacement if a product quality issue occurs within the warranty timeframe. Brace should not be exposed to moisturizers, chemicals, or solvents, as they may affect durability.

APPLICATION INFORMATION:

SIZING:

1. Begin by taking a circumferential measurement of the patient approximately 1" (2.5cm) above the widest part of the hips. For patients whose circumferential measurement is between sizes, select the larger brace.
2. Match the patient's size to the Sizing Chart below.

SIZE	CIRCUMFERENTIAL MEASUREMENT
S/M	Less than 48" (122 cm)
L/XL	49" - 60" (125 cm - 152 cm)

NOTE: It is not uncommon for the patient to experience post-surgical swelling, followed by weight loss (10-20+ lbs/4.5-9.1 kg). The Postural Extension brace is designed with two (2) adjustable belts which can be easily sized to achieve optimal fit.

PREPARE BRACE:

Prior to applying, un-hook both pull-handles and secure to the side of the belt. Open the belt by un-hooking one Velcro® side panel from the front panel. Loosen both shoulder straps at the buckle (Figure A).

APPLICATION INSTRUCTIONS:

1. Feed one arm through the shoulder strap as if you were applying a backpack; continue with the other arm (Figure B).
2. Position front/anterior panel with the bottom edge aligned with the pubic bone. Secure Velcro side panel to front panel (Figure C).
3. Adjust the shoulder strap length by pulling on the end of the strap. Be sure to maintain the bottom edge of the front panel with the pubic bone and the back panel between the shoulder blades and lower back (Figure D).

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4. With product in proper position, tighten brace by slipping thumbs through the holes in the pull tabs. Pull away from your body. When tightened, secure the pull tabs to the loop exterior. (Figure E)
5. To Remove; release both pull-handles and one side of the belt. Slide the brace off the other shoulder like removing a backpack.

BRACE ADJUSTMENTS:

- **Waist Belt:** If needed, to modify the waist belt, adjust the side panels by un-hooking the Velcro on the back panel and moving panels more center for a smaller waist, or closer to the edge of the back panel for a larger waist.
- **Shoulder Straps:** If needed, the extra length in the straps can be trimmed with scissors.
- **Back Panel:** Length can be adjusted by loosening the two screws.

HOT/COLD PACK (SOLD SEPARATELY)

1. Follow instructions on hot/cold pack. Insert hot/cold pack into pocket and adhere to back panel.

ADJUSTING PULL TAB LENGTH (OPTIONAL)

1. Remove lace spool from the pull tab pocket and wind lace around the spool to shorten the length of the pull tab. Unwind lace from the spool to lengthen the pull tab. (Figure F)
2. Insert lace spool back into pull tab pocket.

INSERT/REMOVE FRONT PANEL

1. Open inside panel pocket. Remove semi-rigid panel. (Figure G)
2. Open inside panel pocket and insert semi-rigid panel.

CLEANING INSTRUCTIONS: Separate the brace's removable REAR PANEL before cleaning.

Hand wash in water (86°F/30°C), using mild soap. Air dry.

NOTE: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

Do NOT iron.

Do NOT tumble dry.

Do NOT bleach.

MATERIAL CONTENTS: Nylon, Polyester, Polyurethane, Polyethylene, Acrylic, PVC, EVA, Acetal, Aluminum, Steel, Synthetic Rubber

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations; the provisions of such local regulations will apply.

**FOR SINGLE PATIENT USE ONLY.
RX ONLY.**

PRODUCT NOT MADE WITH NATURAL RUBBER LATEX.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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