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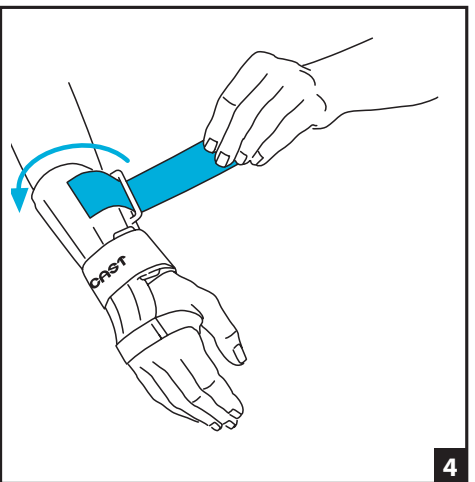
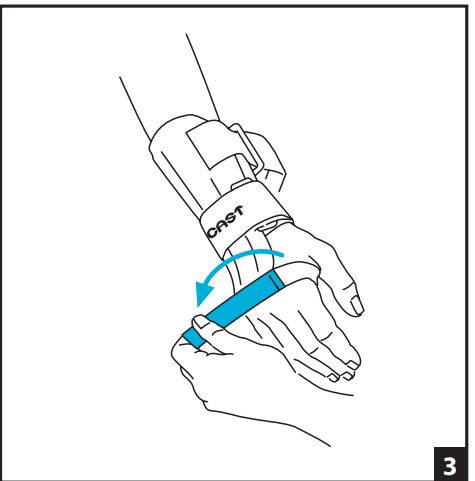
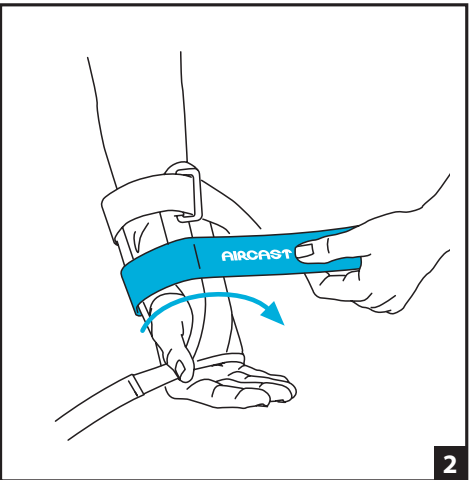
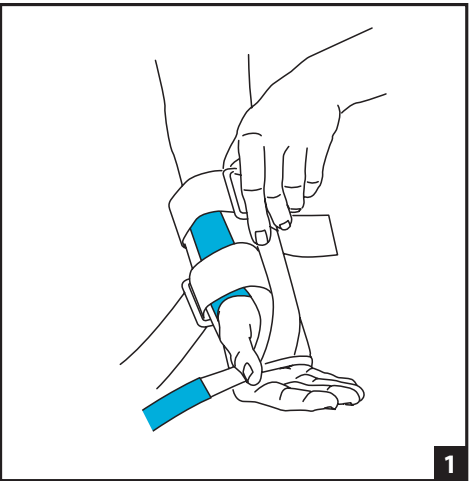
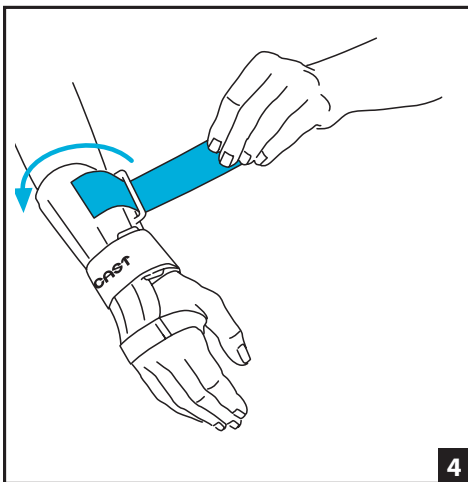
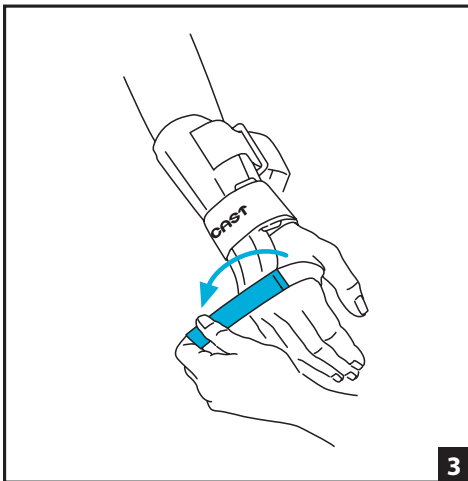
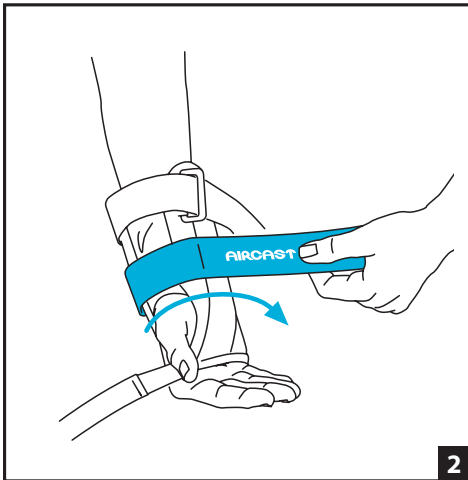
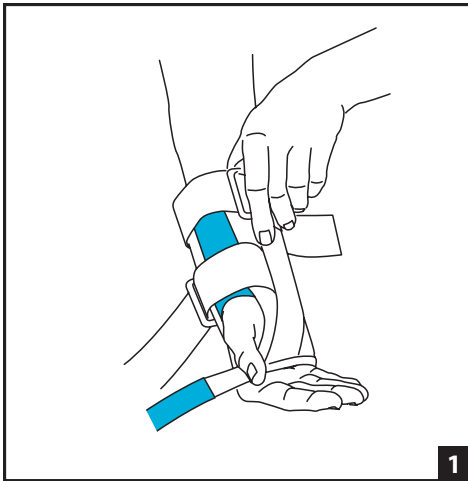
## ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

**INTENDED USER PROFILE:** The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

**INTENDED USE/INDICATIONS:** The Aircast A2 Wrist Brace is designed to provide support for wrist injuries (ligament instability, sprain or muscle strain), carpal tunnel syndrome, post-operative use, and post-removal of casting or splint.

**PERFORMANCE CHARACTERISTICS:** Soft-good/semi-rigid designed to restrict motion through elastic or semi-rigid construction.

**CONTRAINDICATIONS:** N/A

**WARNINGS AND PRECAUTIONS:**

- If you have skin disorders/ injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a medical professional.
- Do not use over open wounds.
- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
- Do not use this device if it was damaged and/or packaging has been opened.

**NOTE:** Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

**APPLICATION INFORMATION:**

1. Unfasten straps. Middle strap and arm strap should remain threaded through black rings on brace.
2. Align brace with green fabric toward thumb and thin strap (hand strap) toward fingers. Insert hand into brace pull brace up wrist until end of brace is just past knuckles (fingers should bend freely at knuckles).
3. Secure brace straps in order: middle strap, hand strap, and arm strap
4. Tighten all straps until snug and comfortable. All straps may be trimmed with scissors as needed.

**CLEANING INSTRUCTIONS:** Hand wash in cold water with mild soap and air dried. **Remove dual stabilizers before cleaning:** Stabilizer opening is on the inside of brace.

**WARRANTY:** DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

**FOR SINGLE PATIENT USE ONLY**

~~NOT~~ **NOT MADE WITH NATURAL RUBBER LATEX.**

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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