**DONJOY®**

**ULTRASLING® IV AND  
ULTRASLING® IV AB**

ULTRASLING® IV Y ULTRASLING® IV AB  
ULTRASLING® IV UND ULTRASLING® IV AB  
ULTRASLING® IV E ULTRASLING® IV AB  
ULTRASLING® IV ET ULTRASLING® IV AB  
ULTRASLING® IV EN ULTRASLING® IV AB  
ULTRASLING® IV E ULTRASLING® IV AB  
ULTRASLING® IV A ULTRASLING® IV AB  
ULTRASLING® IV A ULTRASLING® IV AB  
ULTRASLING® IV OG ULTRASLING® IV AB  
ULTRASLING® IV OCH ULTRASLING® IV AB  
ULTRASLING® IV JA ULTRASLING® IV AB  
ULTRASLING® IV OG ULTRASLING® IV AB  
ULTRASLING® IV 和 ULTRASLING® IV AB  
ULTRASLING® IV および ULTRASLING® IV AB







ENGLISH .....2  
ESPAÑOL .....4  
DEUTSCH .....6  
ITALIANO.....8  
FRANÇAIS.....10  
NEDERLANDS .....12  
PORTUGUÊS .....14  
ČEŠTINA .....16  
SLOVENČINA.....18  
DANSK..... 20  
SVENSKA ..... 22  
SUOMI ..... 24  
NORSK..... 26  
简体中文 ..... 28  
日本語 ..... 30

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DJO, LLC  
5919 Sea Otter Place, Suite 200  
Carlsbad, CA 92010 · USA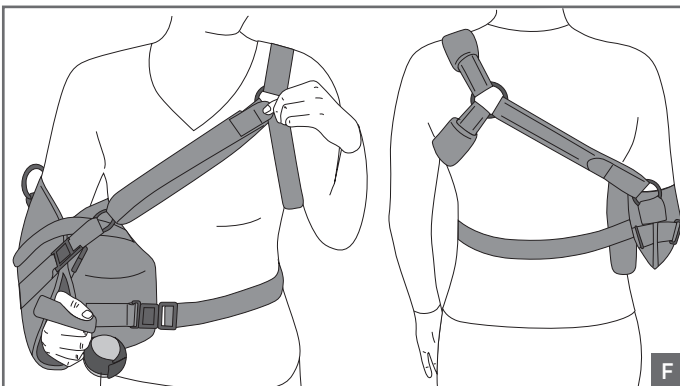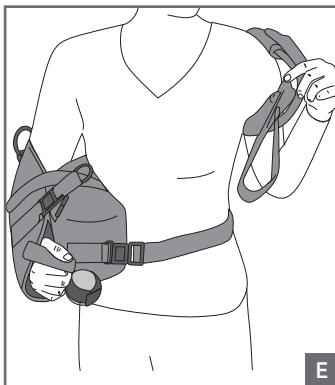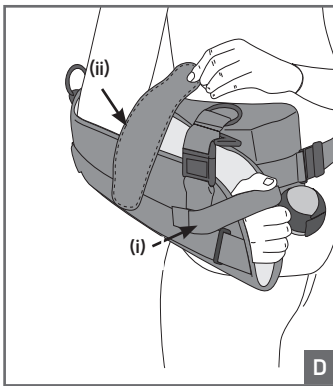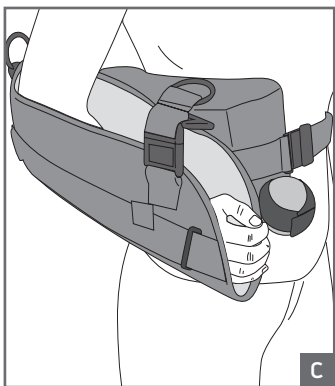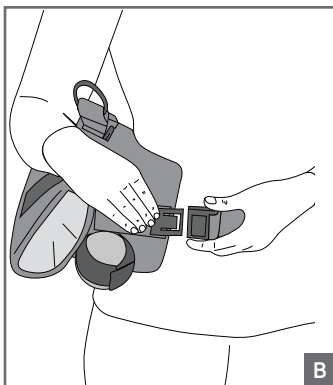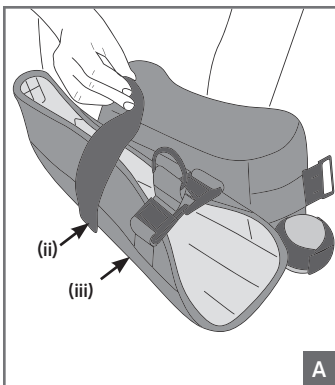

## ENGLISH

### BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

#### INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

**INTENDED USE / INDICATIONS:** Post-operative rotator cuff repairs, Bankart procedures, Capsular shifts, Glenohumeral dislocation/subluxation, Soft tissue strains/repairs.

#### PERFORMANCE CHARACTERISTICS:

- Soft/good/ semi-rigid designed to restrict motion through elastic or semi-rigid construction.
- Providing immobilization or controlled movement of the limb or body segment.

#### CONTRAINDICATIONS: None.

#### WARNINGS AND PRECAUTIONS:

- If you have skin disorders/injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a medical professional.
- Do not use over open wounds.
- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
- Do not use this device if it was damaged and/or packaging has been opened.

**NOTE:** Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

#### APPLICATION INFORMATION:

To ensure proper fit, if possible, product should be sized on patient pre-operatively. Product is shipped with strapping and sling configured for a right shoulder/arm. To convert for left shoulder/arm, detach sling from abductor cushion; flip cushion over keeping waist buckle at front; attach sling to outside of cushion with Velcro strip.

#### ULTRASLING IV:

1. Release waist belt on cushion and forearm strap (ii) on sling. Unfasten the shoulder strap (iii) by releasing the quick release buckles at the front of sling (Fig. A).
2. Align cushion on the injured side at waist/hip level with elbow fixed at 90°. Narrow end of cushion should be positioned to the back of forearm/elbow. Bring waist strap back and attach to buckle at front of pillow (Fig. B). Adjust strap and trim as necessary for proper fit.
3. Place forearm into the sling with arm as far back as possible in the sling (Fig. C). Secure thumb strap (i) between the thumb and forefinger at front of sling. Secure forearm strap (ii) to sling and cushion (Fig. D).
4. Using your free arm, reach behind your body and slip arm through shoulder straps B and C (Fig. E). This motion is similar to putting on a backpack. One strap should be over the unaffected shoulder (clavicular area) and the other under the arm (straps B & C). Adjust and trim both straps for a proper fit.
5. Adjust shoulder strap A through O-ring on front of sling. Adjust ALL straps for adequate stabilization of injured shoulder (Fig. F). Remove Y-tabs on strap ends and trim as needed. Replace Y-tabs and secure straps.

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## ULTRASLING IV AB:

1. Release waist belt on cushion and forearm strap (ii) on sling. Unfasten the shoulder strap (iii) by releasing the quick release buckles at the front of sling (Fig. A).
2. Place the large pillow under the affected area with the 45° angle facing up. Attach the sling to the cushion. Align the hook and loop strip and attach to buckle at the front of pillow (fig B). Adjust strap and trim as necessary for proper fit.
3. Place forearm into the sling with arm as far back as possible in the sling (Fig. C). Secure thumb strap (i) between the thumb and forefinger at front of sling. Secure forearm strap (ii) to sling and cushion (Fig. D).
4. Using your free arm, reach behind your body and slip arm through shoulder straps B and C (Fig. E). This motion is similar to putting on a backpack. One strap should be over the unaffected shoulder (clavicular area) and the other under the arm (straps B&C). Adjust and trim both straps for a proper fit.
5. Adjust shoulder strap A through O-ring on front of sling. Adjust ALL straps for adequate stabilization of injured shoulder (Fig. F). Remove Y-tabs on strap ends and trim as needed. Replace Y-tabs and secure straps.

**FOR O.R. APPLICATION:** Complete application steps 1 – 3 followed by:

Unfasten one end of axilla strap (note: the axilla strap W can be removed if desired). The axilla strap is strap C for the right arm and strap B for the left arm. Bring shoulder strap over the unaffected shoulder. Attach the quick release buckles to the sling. Bring axilla (underarm) strap under the arm and feed through flat O-ring. Secure strap with Velcro tab.

**NOTE:** Large, flat O-rings on front (Pectoral/Clavicle) and back (Scapula) should be positioned for maximum patient comfort.

**FOR REHABILITATION:** To use the UltraSling IV without the Abductor Cushion, remove waist strap from front and back. Then thread the waist strap through the small D-rings located on front and back of the inner side of the sling. Secure strap around waist with Velcro tabs. Trim as needed.

**CLEANING INSTRUCTIONS:** Hand wash in cold water 30°C/85°F. Hang dry.

**MATERIAL CONTENTS:** Nylon, Polyester, Spandex, Polyurethane.

**WARRANTY:** DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

**RX ONLY.**

**FOR SINGLE PATIENT USE ONLY**

**NOT MADE WITH NATURAL RUBBER  
LATEX.**

**NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.**

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